



ASKINA® CALGITROL® AG ASKINA® CALGITROL® THIN

IONIC SILVER ALGINATE DRESSINGS
B. Braun Wound Care. Right. From the start.

Askina® Calgitrol® Ag | Askina® Calgitrol® THIN

Silver alginate dressings



Broad antimicrobial effectiveness ⁽¹⁾

Immediate availability of silver ions ⁽²⁾

Sustained controlled release to the wound bed during use of the dressing ⁽²⁾

Tolerable and antimicrobially efficient ⁽³⁾


Easy to use & conformable ⁽⁴⁾

No activation needed: ready to use

Askina® Calgitrol® Ag is a sterile dressing consisting of two layers :

- an absorbent polyurethane foam layer which provides for the absorption of wound exudate
- an ionic silver alginate matrix which provides for broad antimicrobial effectiveness and helps reduce the bacterial load

Askina® Calgitrol® THIN is a thin layer of ionic silver alginate matrix. Soft and conformable, it is well adapted for deep and difficult-to-dress wounds.

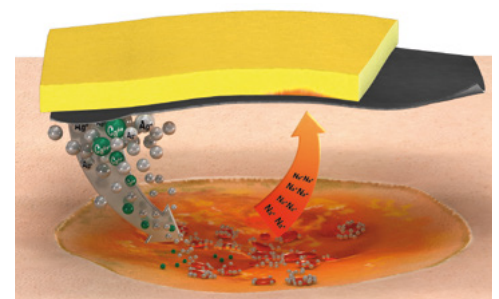


In contact with wound exudate, the Calgitrol® ionic silver alginate matrix forms a soft gel allowing the liberation of silver ions.

(1) Instruction for use: Askina® Calgitrol® Ag, Askina® Calgitrol® THIN, Askina® Calgitrol® Paste.
(2) Opananon S, Magnette A, Meuleniére F, Harding K. Askina® Calgitrol® Made Easy. Wounds International 2012; 3(1). Available from www.woundsinternational.com
(3) Trial C, Darbas H, Lavigne J-P, Sotto A, Simoneau G, Tillet Y, et al. Assessment of the antimicrobial effectiveness of a new silver alginate wound dressing: a RCT. J Wound Care. 2010 Jan;19(1):20-6.
(4) WoundsInternational. Using Askina® Calgitrol® Paste for the treatment of diabetic foot infection: casestudies.London:WoundsInternational2013.Availablefromwww.woundsinternational.com



TIPS FOR APPLICATION

- ▶ Apply the dressing with silver matrix (dark grey surface) touching the wound
- ▶ Secure in place with an appropriate fixation dressing
- ▶ After dressing removal, thoroughly cleanse the wound to remove any residue left from Askina® Calgitrol® Ag / Askina® Calgitrol® THIN

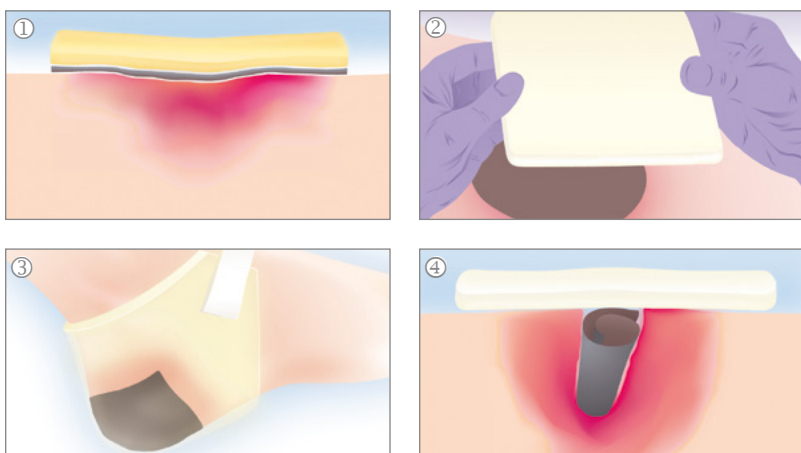

Indications


Askina® Calgitrol® Ag is indicated for the management of exuding partial to full thickness wounds, stage I-IV pressure sores, venous ulcers, second degree burns and donor sites. It is indicated for external use only and may be used in the management of infected wounds under medical supervision at the discretion of the physician.

Examples of use

- ① Use of Askina® Calgitrol® Ag on flat wound
- ② Askina Calgitrol® THIN used for a flat wound with irregular surface with Askina® Foam as a secondary dressing
- ③ Use of Askina Calgitrol® THIN on the heel wound, in combination with Askina® Heel
- ④ Askina Calgitrol® THIN is also suitable for cavity wounds




Askina® Calgitrol® Ag

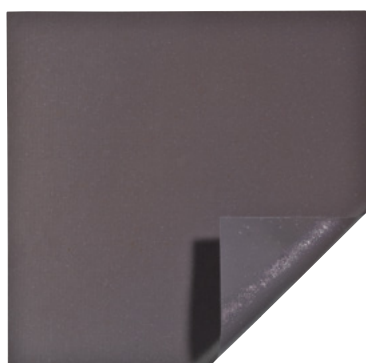

Askina® Calgitrol® THIN

Sustained controlled release of silver ions from Askina® Calgitrol® Ag⁽²⁾

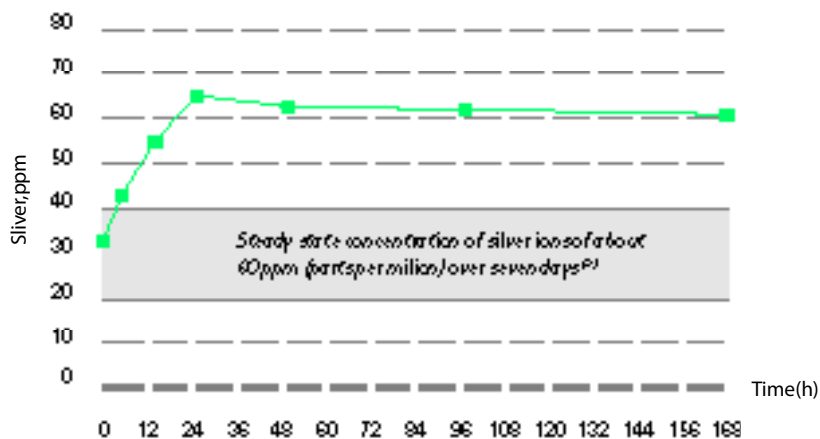

	Size	Pcs/Pack	Article number
Askina® Calgitrol® Ag	10 cm x 10 cm	3 10	6211003 6211010
	15 cm x 15 cm	3 10	6211503 6211510
	20 cm x 20 cm	10	6212010
Askina® Calgitrol® THIN	5 cm x 5 cm	10	6205510
	10 cm x 10 cm	10	6201010
	10 cm x 20 cm	10	6202110
	20 cm x 20 cm	10	6202010
	20 cm x 40 cm	3	6202403
	30 cm x 30 cm	3	6203003
	40 cm x 40 cm	3	6204003

Manufactured by: B. Braun Hospicare Ltd. | Collooney | Co. Sligo | IRELAND
www.bbraun.com

This document, its contents, including institutional data, information, trademarks and logos mentioned herein are the exclusive property of B. Braun. Any representation and / or reproduction, total or partial, of this document and its contents without the express prior consent of B. Braun, is strictly prohibited and constitutes an infringement of the intellectual property rights of B. Braun.

Non binding document and photographs.