



Original frame delivery system

- All-in-one sterile, waterproof dressing
- Picture frame delivery for easy application
- Bacterial and viral barrier reduces risk of infection
- Absorbent, non-adherent pad

Tegaderm™ + Pad Transparent Film Dressing with Absorbent Pad Frame delivery for Easy Application

3587
9cm x 10.5cm - 25 dressings/box
4 boxes/case

3593
9cm x 35cm - 25 dressings/box
4 boxes/case

3591
9cm x 25cm - 25 dressings/box
4 boxes/case

3590
9cm x 20cm - 25 dressings/box
4 boxes/case

3589
9cm x 15cm - 25 dressings/box
4 boxes/case

3586
9cm x 10cm - 25 dressings/box
4 boxes/case

3584
6cm x 10cm - 50 dressings/box
4 boxes/case

3582
5cm x 7cm - 50 dressings/box
4 boxes/case

Reasons for choosing 3M™ Tegaderm™ + Pad Film Dressing with Non-Adherent Pad

- All-in-one sterile dressing – reduces application time
- Bacterial and viral barrier* reduces infection risk
- Waterproof backing allows patient to shower
- Picture frame delivery system ensures precise placement without applying tension
- Transparent film allows monitoring of pad absorption and surrounding skin
- Conforms to body contours, providing comfort
- Breathable film allows skin to function normally with good exchange of moisture, vapour and oxygen
- Full range of sizes
- Provides moist wound environment to enhance healing and lessen pain

Indications for Use

- Surgical wound dressing
- Small incisions
- Lacerations and abrasions
- Superficial and partial thickness burns
- Light to moderate draining wounds
- As a secondary dressing

Application Technique



Remove the back liner. Centre the dressing over the wound and apply the dressing without tension from the centre outwards



Remove the frame



Press firmly around the dressing to activate adhesive to the skin

* Laboratory testing has proven that the film provides a barrier against HIV-1 and HBV while the dressing remains intact without leakage. Data on file