

Tubigrip®

Multi-purpose tubular bandage

Tubigrip provides tissue support in the treatment of strains and sprains, soft tissue injuries, general oedema, post- burn scarring and ribcage injuries and is also used for pressure dressings and arm fixation.



About Tubigrip

Tubigrip provides lasting, effective support with complete freedom of movement for the patient. Once the bandage has been applied, covered elastic threads within the fabric move to adjust to the contours of the body and distribute pressure evenly over the surface. Fitting is quick and easy.

Tubigrip stays securely in position without pins or tapes, unlike conventional crepe bandages. As it can easily be reapplied after washing, without loss of effect, Tubigrip is particularly useful for outpatients and in the community.

Tubigrip reduces wastage, as only the exact amount required is cut from the roll. Apply with a 2–3cm overlap.

Tubigrip is available in two shades – natural and beige.

Tubular bandage applicators

Tubular Bandage Applicators may be used to make application easier over bulky dressings or painful joints.

Tubigrip is loaded onto the applicator, which is then positioned over the affected area. The end of the bandage is held in position while the applicator is gently removed, leaving the Tubigrip in place. Form a double layer by doubling back the bandage. Ensure that the upper edge of the second layer is taken 2–3cm higher up the limb than the first.

The applicators are available in a full range of sizes and are made from durable chrome-plated steel.

How to apply Tubigrip



1. Cut Tubigrip to twice length required for limb, allowing an extra 2–3cm for overlap.



2. Pull Tubigrip onto limb like a stocking.



3. Double Tubigrip back over limb. Ensure upper edge is taken 2–3cm higher up the limb than the first.

The benefits of using Tubigrip

- Provides comfortable, effective tissue support
- Easy to apply and reapply
- Full range of sizes to suit most applications
- No pins or tapes needed
- Washable and reusable
- Choice of two colours for patient acceptability

		Natural Shade		Beige Shade	
		1m code	10m code	1m code	10m code
A	Infant legs and arms		1435		
B	Small wrists	1520	1436		1449
C	Medium wrists, Small elbows		1443	1545	1450
D	Large wrists, Medium elbows, Small/Medium ankles, Small knee		1437	1546	1451
E	Large elbows, Medium/Large ankles, Medium knee		1434	1547	1448
F	Large knee, Small thigh		1438	1548	1452
G	Xlarge Knee, Medium thigh	1524	1439		1453
J	Large Thigh, Small trunk		1440		
K	Small/Medium Trunk		1441		
L	Large Trunk		1442		
M	Xlarge trunk		1444		