




Even **better** than you think

Self-adhesive absorbent dressing

Mepore is an air permeable, self-adhesive dressing ideal for low to moderately exuding wounds.

Its skin friendly water based, solvent free polyacrylate adhesive provides a gentle and secure fixation.

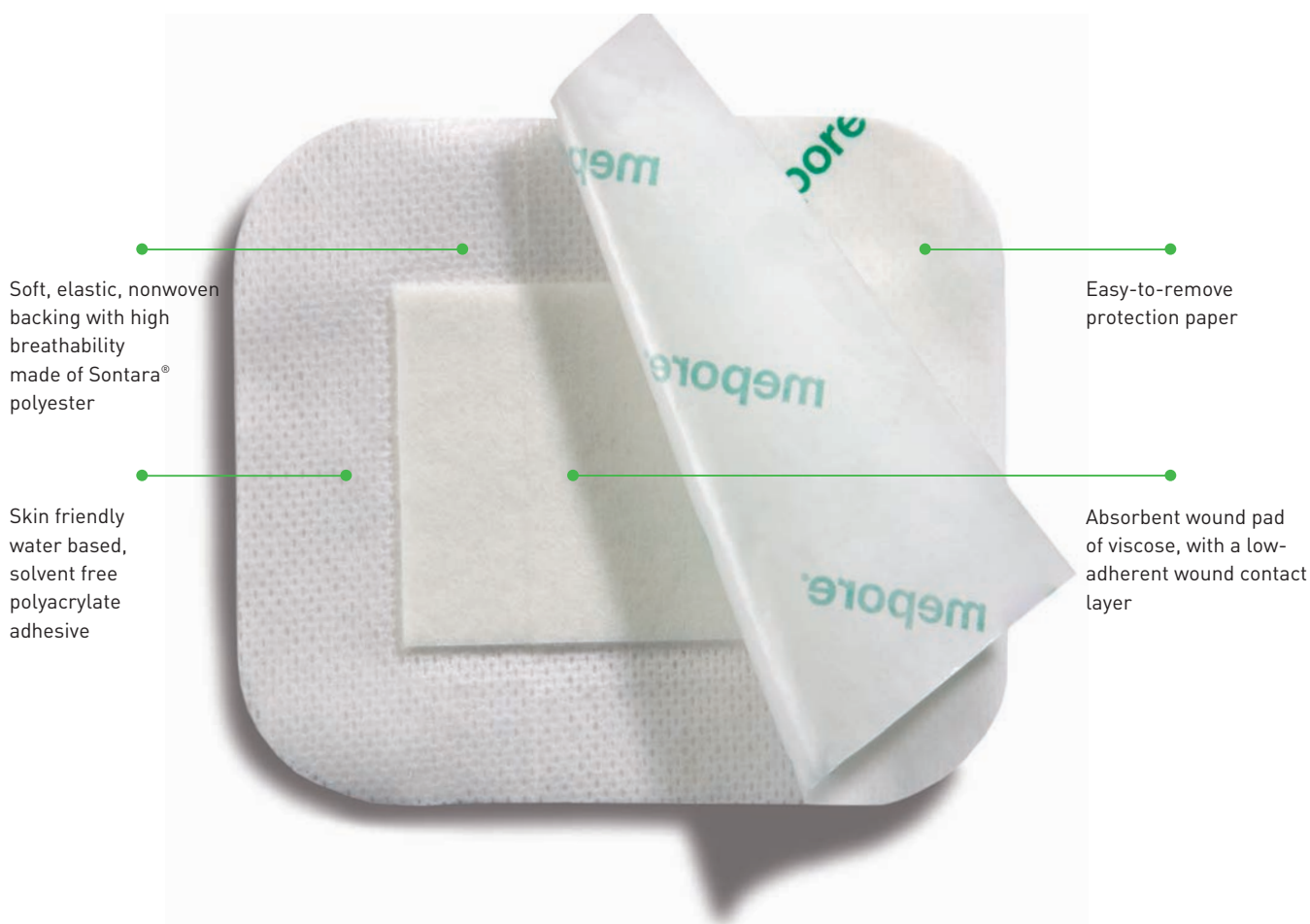

Soft, elastic, nonwoven backing with high breathability made of Sontara® polyester

Skin friendly water based, solvent free polyacrylate adhesive

Easy-to-remove protection paper

Absorbent wound pad of viscose, with a low-adherent wound contact layer

How Mepore works

The porous structure of the Sontara® nonwoven fabric ensures air and water vapour permeability. The fabric is flexible, conforms well to body contours and allows body movements when in place. The fabric is flexible and non-absorbent allowing freedom of movement and a barrier for blood and fluid strike-through. The polyacrylate adhesive has been designed to be gentle to the skin and at the same time provide secure fixation. The wound pad

combined with a low-adherent wound contact layer absorbs moderate amounts of wound fluid and reduces the risk of adherence to the wound surface. The easy-to-remove protection paper permits sterile application, as well as simplifying dressing application.

Areas of use

Mepore can be used in a wide variety of wounds which have low to moderate exudate levels. These include surgical wounds, minor burns, cuts and abrasions.

How to use Mepore



Open the package and remove the dressing.



Grip both overlapping release papers, exposing adhesive surface allowing initial fixation of the dressing.



Position the dressing on the skin without stretching and gently remove the rest of the release papers. Firmly smooth adhesive border to obtain proper adhesion. Do not stretch the dressing when applying.

Frequency of change: The frequency of dressing changes will be governed by the condition of the underlying wound. Upon removal grasp the edge and slowly peel the dressing from the skin in the direction of hair growth.

Benefits of Mepore

- Skin friendly water based adhesive
- Low adherent wound contact layer minimises risk of adherence to the wound
- Flexible fluid repellent backing
- Easy-to-remove protection paper allows for convenient and aseptic application
- Air permeable, prevents maceration and provides comfort
- Rounded corners for secure fixations (all sizes up to 9 x 15 cm)

Mepore is packaged sterile in single packs

Art. No	Size cm	pcs/shelf cont.	pcs/transp. cont.
670800	6 x 7	60	480
670900	9 x 10	50	400
671000	9 x 15	50	400
671100	9 x 20	30	180
671200	9 x 25	30	180
671300	9 x 30	30	270
671400	9 x 35	30	180

Precautions

Care should be taken to ensure that Mepore is not applied under tension, to prevent shearing force. This is particularly important when the dressing is applied over joints.

