

Mepore[®] Pro

Even **better** than you think

Showerproof, self-adhesive absorbent dressing

Mepore pro is a self-adherent, absorbent and breathable dressing with an outer film layer which protects the wound from water and contamination. Its skin-friendly water based, solvent free polyacrylate adhesive provides a gentle and secure fixation.



How Mepore pro works

The backing film allows showering and protects the wound from water and contamination. It also protects clothing and bedlinen from blood or exudate strike-through. Combined with the highly absorbent pad and the high vapour permeability of the backing paper, Mepore Pro offers excellent fluid handling.

The polyacrylate adhesive is designed to be gentle to the skin but at the same time provide secure fixation.

The easy-to-remove protection paper permits sterile application, as well as simplifying dressing application. The stability of the fabric/film laminate also supports ease of application.

Areas of use

Mepore pro can be used in a wide variety of wounds which have low to moderate exudate levels. These include surgical wounds, minor burns, cuts and abrasions.

How to use Mepore pro



Open the package and remove the dressing.



Grip both overlapping release papers, exposing adhesive surface allowing initial fixation of the dressing.



Position the dressing on the skin without stretching and gently remove the rest of the release papers. Firmly smooth adhesive border to obtain proper adhesion. Do not stretch the dressing when applying.

Frequency of change: The frequency of dressing changes will be governed by the condition of the underlying wound. Upon removal grasp the edge and slowly peel the dressing from the skin in the direction of hair growth.

Benefits of Mepore pro

- Allows showering
- Low adherent wound contact layer minimizes risk of adherence to the wound
- Protects wound from water and outside contamination
- Viral and bacteria proof backing film
- Optimized absorption capacity
- Easy-to-remove protection paper allows for convenient and aseptic applications
- Breathable film
- Low friction to clothing and bedlinen
- Skin-friendly water based adhesive

- Rounded corners for secure fixation (all sizes up to 9 x 15 cm)

Precautions

Care should be taken to ensure that Mepore pro is not applied under tension, to prevent shearing forces causing damage to the skin. This is particularly important when the dressing is applied over joints.



Mepore pro is packaged sterile in single packs.

Art. No	Size cm	Pcs/ shelf cont.	Pcs/ transp. cont.
670820	6 x 7	60	480
670920	9 x 10	40	320
671020	9 x 15	40	320
671120	9 x 20	30	180
671220	9 x 25	30	180
671320	9 x 30	30	270