

Soothing and hydrating
gel that promotes healing

 **smith&nephew**
SOLOSITE[®]
Preserved Multi-Use Hydrogel



SOLOSITE[◇] gel is suitable for debriding and desloughing wounds at the earliest stages of healing

SOLOSITE gel promotes early healing

- Provides a moist environment at the wound surface
- Prevents eschar formation which may aid in fast healing

Debriding action

Any dead tissue in a wound must be removed before healing can proceed, so careful cleansing and debridement is essential in the initial management of open wounds. SOLOSITE gel is easily applied and removed, and by its gentle yet effective re-hydrating action, can assist debridement of dead tissue.

By maintaining moisture content at the wound surface, SOLOSITE gel helps to promote a moist healing environment.

Desloughing action

In the inflammatory phase of the wound healing process, slough is formed by the accumulation of dead cells within the wound exudate.

Where slough has already formed, the gel's re-hydrating action gently yet effectively removes it without damaging healthy tissue.

Application

SOLOSITE gel requires a secondary dressing (e.g. OPSITE[®] Post-Op) to hold it in place. The frequency of dressing change recommended depends on the amount of fluid produced by the wound, but it is usually every 1 to 3 days. Irrigate with normal saline to remove gel from the wound bed. SOLOSITE gel should be removed if leakage of fluid occurs past the secondary dressing.

Indications

SOLOSITE gel is used to create a moist wound environment for the treatment of conditions such as:

- Pressure injuries and leg ulcers
- Lacerations
- Abrasions
- Minor burns

SOLOSITE gel is suitable where dressing changes are to be carried out more frequently and a multi-use presentation is more convenient.



Ordering information

Code	Product description	Items per unit
36100614	20g Tube	Box/10
36361354	50g Tube	Box/12
449600	85g Flip-top Tube	Box/12

Supporting healthcare professionals for over 150 years

Smith & Nephew Pty Ltd
Australia
T +61 2 9857 3999 F +61 2 9857 3900
www.smith-nephew.com/australia
Customer Service
T 13 13 60 F 1800 671 000

Smith & Nephew Ltd
New Zealand
T +64 9 820 2840 F +64 9 820 2841
www.smith-nephew.com/new-zealand
Customer Service
T 0800 657 799 (Surgical) T 0800 807 663 (Wound)

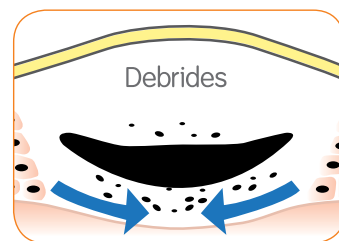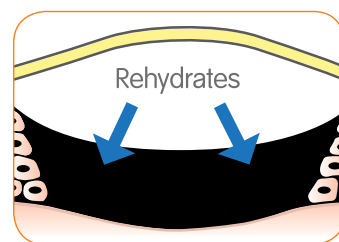

Please consult product labels and inserts for any indications, contraindications, hazards, warnings, cautions and instructions for use.

◇Trademark of Smith & Nephew
SN12764 (11/2016)