

Application guide

IODOSORB Cadexomer Iodine Powder promotes wound bed preparation by removing barriers to healing from the wound.*¹⁻⁸ With a simple and easy application and removal,^{1,6} IODOSORB Dressing is clinically proven to treat infection in chronic wounds.^{1,8-11}

Applying IODOSORB Powder



- Wash hands thoroughly before and after use
- Using gloved hands, clean the wound and the surrounding area with either a gentle stream of water or saline
- **DO NOT DRY** the wound surface



- Apply IODOSORB Powder to the wound surface to form a layer 3mm deep, which conforms to the shape of the wound



- Cover the wound completely with an appropriate secondary dressing.
- Apply compression bandaging if appropriate
- Any remaining IODOSORB Powder should be discarded due to loss of sterility

Changing IODOSORB Powder



- IODOSORB Powder should be changed when it has become saturated with wound fluid and all the iodine has been released
- This is indicated by loss of colour, usually two to three times a week
- If the wound is discharging heavily, daily changes may be needed



- If necessary, soak the dressing for a few minutes, then remove
- Gently remove the IODOSORB Powder using a stream of water or saline
- Gently blot any excess fluid, leaving the wound surface slightly moist, before reapplying IODOSORB Powder



- Gently blot any excess fluid, leaving the wound surface slightly moist, before reapplying IODOSORB Powder

*e.g. slough, pus, debris, and biofilm.

Indications: Assists the healing and treatment of chronic ulcers. IODOSORB reduces the bacterial count, facilitates desloughing, absorbs exudate and maintains a moist wound environment to promote healing.

Contraindications: Not to be used in patients with known or suspected iodine sensitivity.

Precautions: Should not be used in children under 12 years of age.

This material is intended for healthcare professionals. For detailed product information, including indications for use, contraindications, precautions and warnings, please consult the product's Instructions for Use (IFU) prior to use.

Smith & Nephew Pty Ltd
Australia
T +61 2 9857 3999
F +61 2 9857 3900
smith-nephew.com/en-au

Smith & Nephew Ltd
New Zealand
T +64 9 820 2840
F +64 9 820 2841
smith-nephew.com/en-nz

[◇]Trademark of Smith+Nephew
All trademarks acknowledged
© July 2023 Smith+Nephew
39439-anz V1 07/23



IODOSORB Cadexomer Iodine Powder

Code	Size	Pack
66051070	7 x 3g	7

IODOSORB[◇]
Cadexomer Iodine Powder

References: 1. Skog E, Arnesjo B, Troeng T, et al. *British Journal of Dermatology*. 1983;109:77-83. 2. Troeng T, Skog E, Arnesjo B, et al. Stuttgart: Schattauer Verlag; 1983. 3. Holloway GA, Johansen KH, Barnes RW, Pierce GE. *The Western Journal of Medicine*. 1989;151(1):35-38. 4. Malone M, Johani K, Jensen SO, et al. *J Antimicrob Chemother*. 2017;72(7):2093-2101. 5. Hansson C, Persson L-M, Stenquist B, et al. *International Journal of Dermatology*. 1998;37(390-396). 6. Moberg S, Hoffman L, Grennert M-L, Holst A. *Journal of American Geriatrics Society*. 1983;31(6):462-465. 7. Harcup JW, Saul PA. *The British Journal of Clinical Practise*. 1986;40(9):360-364. 8. Lantis JC. Paper presented at: WUWHs; 2016; Florence. 9. Hillstrom L. *Acta Chir Scand Suppl*. 1988;544:53-56. 10. Schwartz JA, Lantis JC 2nd, Gendics C, et al. *Int Wound J*. 2013;10(2):193-199. 11. Malone M, Schwarzer S, Radzieta M, et al. *Int Wound J*. 2019;16(6):1477-1486.